Actreen® Glys Cath

GB

Directions for use

Actreen® Glys Cath is a ready-to-use sterile catheter indicated for intermittent urinary catheterisation.

- ♀ In women, the urethra is short, about 4 cm long. Carefully locate the urethral opening, using a mirror at first.
- ♂ In men, the urethra is longer and has curves that make catheterisation a little more difficult.

Wash your hands very carefully. Make yourself comfortable.

- ♀ Sit or stand, with the legs well apart in order to locate the urethral opening.
- ♂ Sit or stand.

1 Wash the genitourinary area.

- ♀ Take care to wash from the front to the back, beginning with the greater lips of the vagina and ending with the urethral opening.
- ♂ First pull back the foreskin completely. Wash carefully both sides of the glans and then the urethral opening.

2 Peel open the outer packaging and remove the catheter.

3 Remove the protective sleeves from around the catheter. You should now be left with just the sheath surrounding the catheter itself.

4 Unfold the top of the sheath and tear open on the perforated line.

5 The catheter is sterile and lubricated. Push the tip of the catheter out of the sheath a short distance. Do not touch the tip. You can hold the catheter through the sheath.

6 If you are not close to a toilet, connect Actreen Cath to a urine collection bag.

- ♀ Once the urethral opening has been located, introduce the catheter until it reaches the bladder.
- ♂ Hold the penis upwards (1) to decrease curvature and facilitate catheter insertion. In order to correctly guide the Actreen® Glys Tiemann catheter through the curves of the male urethra, please align the mark on the connector appropriately. Once the catheter has been correctly positioned (2) the urine will drain.

7 The urine drains. Hold the catheter in place until the bladder has completely drained. Manual pressure on the lower abdomen towards the end of catheterisation will ensure that the bladder has been completely emptied. When the bladder is completely empty, gently withdraw the catheter. If you have used a urine collection bag, disconnect it from the catheter and empty.

8 Discard the disposable catheter and wash hands.

NB : Self-catheterisation should only be carried out under medical advice and on the order of a physician and only in accordance with the instructions given.

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GB – Warning: Contains DEHP. Animal experiments have shown DEHP to be potentially toxic to reproduction. Given the current state of scientific knowledge, a risk (in case of long-term use) especially for premature male babies cannot be completely excluded. As a precaution, the use of medical products containing DEHP should be restricted to short-term use for pregnant women, breastfeeding mothers, infants, and children.

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