Actreen® Lite Cath

- EN** Instructions for use
- DE** Gebrauchsanweisung
- ES** Instrucciones de uso
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing
- CS** Návod k použití
- DA** Brugsanvisning
- EL** Οδηγίες χρήσεως
- FI** Käyttöohje
- HU** Használati utasítás
- PL** Instrukcja użycia
- PT** Instruções de utilização
- RU** Инструкция по применению
- SK** Návod na použitie
- SV** Bruksanvisning
- TR** Kullanım Kılavuzu

B | BRAUN


Product range

		Charrière (Ø mm)	References	References
Actreen® Lite Tiemann 45 cm, 1x30 pcs 1x60 pcs** 		CH08 (2,7 mm)	228108*	228108D**
		CH10 (3,3 mm)	228110*	228110D**
		CH12 (4,0 mm)	228112*	228112D**
		CH14 (4,7 mm)	228114*	228114D**
		CH16 (5,3 mm)	228116*	228116D**
		CH18 (6,0 mm)	228118*	228118D**
Actreen® Lite Nelaton 45 cm, 1x30 pcs 1x60 pcs** 		CH08 (2,7 mm)	228208*	228208D**
		CH10 (3,3 mm)	228210*	228210D**
		CH12 (4,0 mm)	228212*	228212D**
		CH14 (4,7 mm)	228214*	228214D**
		CH16 (5,3 mm)	228216*	228216D**
		CH18 (6,0 mm)	228218*	228218D**
Actreen® Lite Nelaton 20 cm, 1x30 pcs 1x60 pcs** 		CH06 (2,0 mm)	228306*	228306D**
		CH08 (2,7 mm)	228308*	228308D**
		CH10 (3,3 mm)	228310*	228310D**
		CH12 (4,0 mm)	228312*	228312D**
		CH14 (4,7 mm)	228314*	228314D**
		CH16 (5,3 mm)	228316*	228316D**


* For each country a specific letter is added at the end of the reference

** Available only in Spain

Actreen® Lite Cath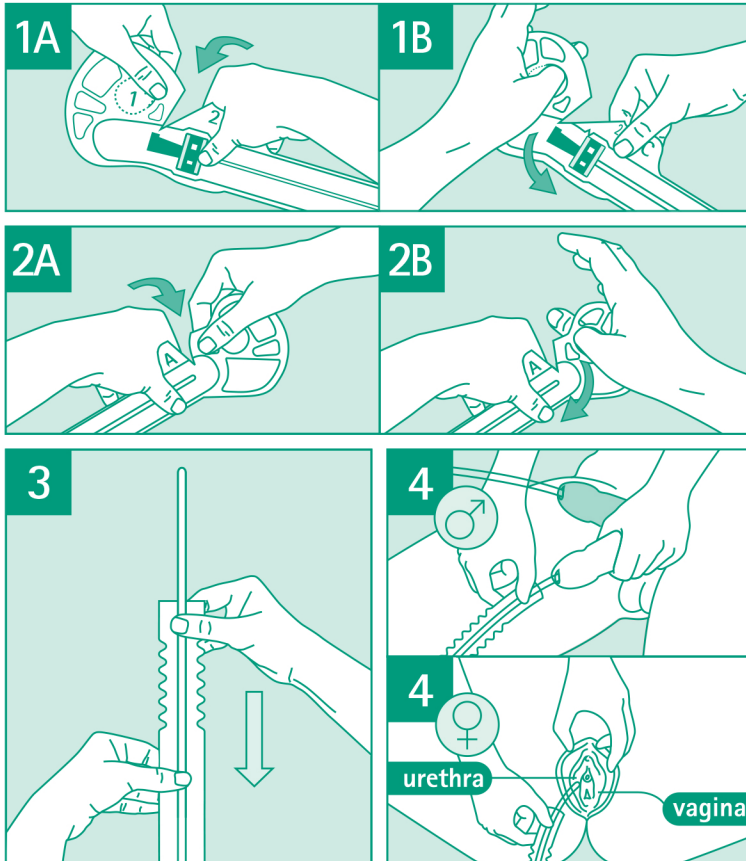

Actreen® Lite Cath

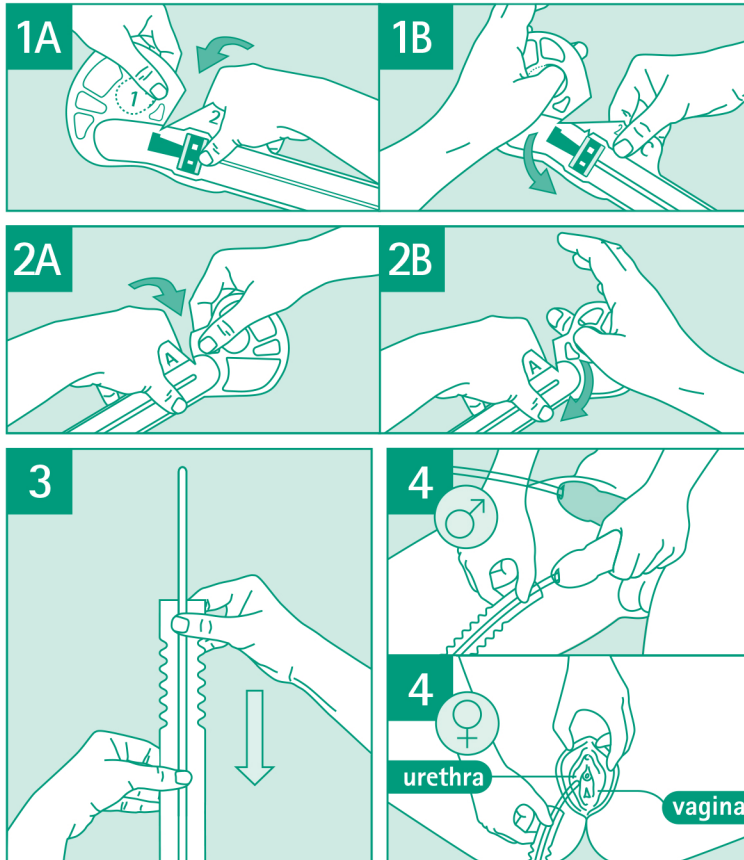
Actreen® Lite Cath

Instructions for use

EN

Actreen® Lite Cath is a ready-to-use sterile pre-lubricated catheter indicated for intermittent urinary catheterization.

Actreen® Lite Cath

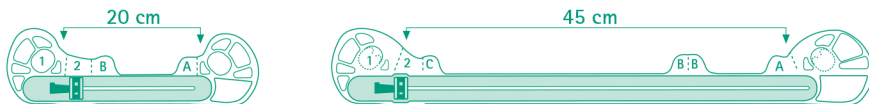Wash your hands very carefully before and after catheterization, and wash the genitourinary area.

Preparation

1. Remove the protective sheath from the connector. There are two ways:

- a – If you have good dexterity, grip and tear off the film at mark "2" level. **1A**
- b – If you have reduced dexterity, with one hand, insert thumb in the eyelet and with the other hand, hold the catheter at mark "2" level. Draw the eyelet downwards and remove the protective sheath from the connector. **1B**

2. Release the tip of catheter:

• «No touch» system:

- a – If you have good dexterity, grip and tear off the sheath between the thumb and the fore-finger at mark "A". **2A**
 - b – If you have reduced dexterity, with one hand, insert thumb in the eyelet and with the other hand, hold the catheter at mark "A". Draw the eyelet downwards and remove the protective sheath from the tip. **2B**
- Fold up the sheath into accordion folds without touching the catheter. **3**

• If you want to release more of the catheter:

- Catheter version 45 cm: Grip and tear off the raised tongue identified by the mark "B" to release 1/3 of the catheter
Grip and tear off the raised tongue identified by the mark "C" to release the full catheter
- Catheter version 20 cm: Grip and tear off the raised tongue identified by the mark "B" to release the full catheter

Catheterization:

Actreen® Lite can be connected to a urine collection bag like Urimed® Bag from B.Braun.

Male:

With one hand hold the penis upwards to decrease curvature of the urethra. With the other hand carefully introduce the catheter into the urethra until it reaches the bladder and the urine drains. **4** ♂ Tiemann catheter: In order to correctly guide the catheter, please align the mark on the connector appropriately.

Female:

With one hand spread the greater lips of the vagina to locate the urethral opening. With the other hand carefully introduce the catheter into the urethra until it reaches the bladder and the urine drains. **4** ♀

Manual pressure on the lower abdomen towards the end of catheterization will ensure that the bladder has been completely emptied. When the bladder is completely empty, gently withdraw the catheter after having clamping it at connector level. The catheter is then discarded according to the current local regulations.

NB: Self-catheterization should only be carried out under medical advice and only in accordance with the instructions given

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B. Braun Medical AB
B. Braun Medical SAS