en DESCRIPTION OF THE PRODUCT: Linovera® consists of hyperoxygenated fatty acids and vegetable extracts.

COMPOSITION: Hyperoxygenated Fatty Acids from sunflower oil. Aloe Barbadensis. Asian Centella. Rosemary Essence.

INDICATIONS AND PROPERTIES: Linovera® is indicated for prevention and treatment of stage I pressure ulcers. Hyperoxygenated essential fatty acids (HOFA) present in the composition of Linovera® favor blood microcirculation, reducing the risk of ischemia and facilitating the renewal of epidermal cells, increasing the resistance of the skin against the agents that cause pressure ulcers. At the same time they protect against friction, reduce skin fragility and prevent dehydration of the skin. In addition, thanks to their antioxidant activity, they reduce the damage caused by free radicals. Linovera® also contains Aloe Barbadensis and Asian Centella. Aloe Barbadensis, also known as Aloe Vera, helps condition the skin and acts as emollient and humectant. Asian Centella also acts as a skin conditioner and has a toning and soothing effect on the skin.

INFORMATION FOR USE: Linovera® is easy to apply and does not require equipment or special knowledge. a) Prevention of stage 1 pressure ulcers: Clean the risk surface or the ulcer. Spray the product on the cleaned surface and massage with the fingertips gently, in order to favor the homogeneous distribution and the absorption. Apply the product 2 - 3 times daily, according to the treatment regimen by the health personnel. b) Treatment of stage I pressure ulcers: Clean the ulcer with a sterile solution. It is recommended the use of a saline solution (0.9%) or a solution for the wound cleansing as Prontosan®. Dry carefully the surrounded skin of the cutaneous wound. Spray Linovera® directly onto the skin. Cover the area with suitable dressings as Askina® and/or bandages to facilitate the wound healing. Apply the product 2 - 3 times daily, according to the treatment regimen by the health personnel.

WARNINGS AND PRECAUTIONS FOR USE: Store the product in the original pack. Protect the container from direct sunlight. Wash your hands before and after using the product. The use of gloves is recommended. External use. Do not ingest. Avoid contact with the eyes. The product is indicated for adults. Do not use after the expiry date indicated in the bottle and the box.

CONTRAINDICATIONS: Do not use in case of known hypersensitivity to any of the ingredients exists.

ADVERSE REACTIONS: Cutaneous manifestations may appear in patients allergic to any of the components. In principle, they are not serious and cease when the application of the product is interrupted or with the use of antihistamines and/or corticoids.

STORAGE: Do not cool. Do not freeze. Dispose of the product in accordance with local regulations.

PRESENTATION: Linovera® is available in different volumes and presentations.

Product	Container	Product	Container
Linovera®	10 units de 10 ml	Linovera®	10 units de 30 ml
Linovera®	10 units de 20 ml	Linovera®	10 units de 50 ml
Linovera®	1 unit de 30 ml		

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