

# Handling Instructions Sterican® Standard



Carefully remove Sterican® from the packaging.



Attach the needle securely to any Luer Lock or Luer Slip syringe by twisting the syringe clockwise while holding the needle close to its base.



Remove the protective cap carefully straight from the needle hub to avoid exposure to the sharp needle point.



Perform injection according to your established procedure.



Take care during disposal as sharp needle bevel is not protected.



Dispose the used needle immediately after application into an approved sharps container following the standard procedures of your facility and /or according to applicable regulations for sharps disposal.